

KAOLII

Pelvic Floor

# Internal Massager



## DESCRIPTION:

The Kaolii massager is an internal (intra-vaginal) massager made of porcelain. It was designed to help physiotherapists and their patients in physiotherapy treatment's specific to the pelvic floor area. The massager can also be used by pregnant women and new moms after childbirth for relaxation exercises or strengthening the pelvic floor, as well as perineal massages. In short, **the masseur is recommended, among other things, for postpartum perinatal and oncology care, treatments for vaginal pain, endometriosis and vaginal atrophy.**

The porcelain used to create the Kaolii massager is imported directly from Limoges in France, as it is one of the highest qualities of porcelain. Since our porcelain is fired at over 1400°C, the surface of the massager becomes completely vitrified and non-porous. This is why porcelain is one of the healthiest materials for the body and therefore very suitable for physiotherapy, perinatal treatments or any other therapeutic interaction with the human body.

The porcelain used for our internal massager is also **hypoallergenic** and **biocompatible**, making this one a great choice for women who have recently given birth, are doing pelvic rehab, or have faced a condition that affects vaginal tissues, such as menopause or cancer treatments.

## CHARACTERISTICS:

- Plastics and Phthalates Free - 100% Body Safe Material;
- Non-porous;
- Compatible with any lubricant type;
- Easy to clean;
- Can be disinfected in boiling water or a kitchen oven;
- Soft and matte surface - perfect for people that are hypersensitive;
- Can be cooled to treat pain;
- Can be warmed to relax muscles;
- Can be used for perineal massages;
- Waterproof;
- 5-year warranty;
- Vibrations free & can transmit vibrations;
- Hand made and assembled in Montreal, Canada.

## HELPFUL FOR:

- Levator Ani Syndrome
- Trigger Points
- Vaginal Pain
- Menstrual Pain
- Endometriosis
- Hemorrhoids
- Postpartum & Prenatal Care
- Oncology Care
- And more...

## TECHNICAL SPECIFICATIONS

- **Material** : Porcelaine de Limoges
- **Composition** : 20-25% du Dalia is hollow for technical reasons
- **Finish**: Matte
- **Color** : White

## EACH BOX INCLUDE:

- 1x discreet and elegant storage box
- 1x Internal Massager
- 1x informative booklet
- 1x organic cotton storage bag
- 1 x certificate of authenticity

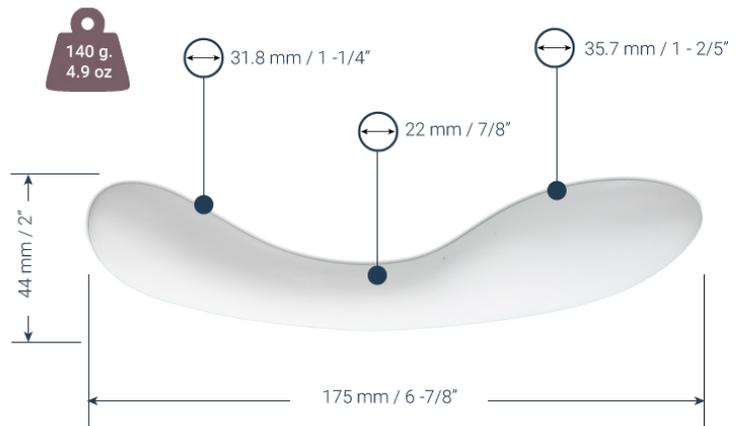


Fig. 1 Kaolii's Internal Massager Dimensions and Weight

## FAQ:

### ***Is porcelain breakable?***

Actually ... yes and no! Porcelain is a material with a very high mechanical strength. It is only fragile to shocks, for example, if there are drastic temperature changes (from freezing cold to very hot) or if it suffers a physical shock (drop on a concrete ground).

\*NB: If for any reason you break one of your porcelain products, you can refer to our Warranty Policy.

### ***Will the internal massager be cold in use?***

Porcelain is a thermally conductive material, so it will be at room temperature when you want to use it. To warm it up, play with it for a few minutes or pass it under hot water.

### ***How can I use it with more than one patient a day?***

If you plan on using the Internal Massager on more than one patient a day, we recommend that you use a condom on it for each use. This will simplify greatly your day and you'll be sure it's perfectly sanitary for each patient.

### ***How to clean my internal massager?***

It's really easy! Just use a mild soap and rinse with water.

You can also disinfect it in boiling water or in a kitchen oven. We haven't tried to sterilize it in an autoclave but we are working on it. \* We do not recommend the use of a dishwasher.

### ***Which lubricant to use with the internal massager?***

Since it's made of porcelain, the internal massager is compatible with all types of lubricants whether water-based or silicone-based. We recommend, however, that you use one that is vegan / organic.